



Instructions for use and reuse of precut mask filters with the Montana Mask

Compliant with the "Decontamination & Reuse of Filtering Facepiece Respirators" issued by the CDC. See <https://www.cdc.gov/coronavirus/2019-ncov/hcp/ppe-strategy/decontamination-reuse-respirators.html> for more information.

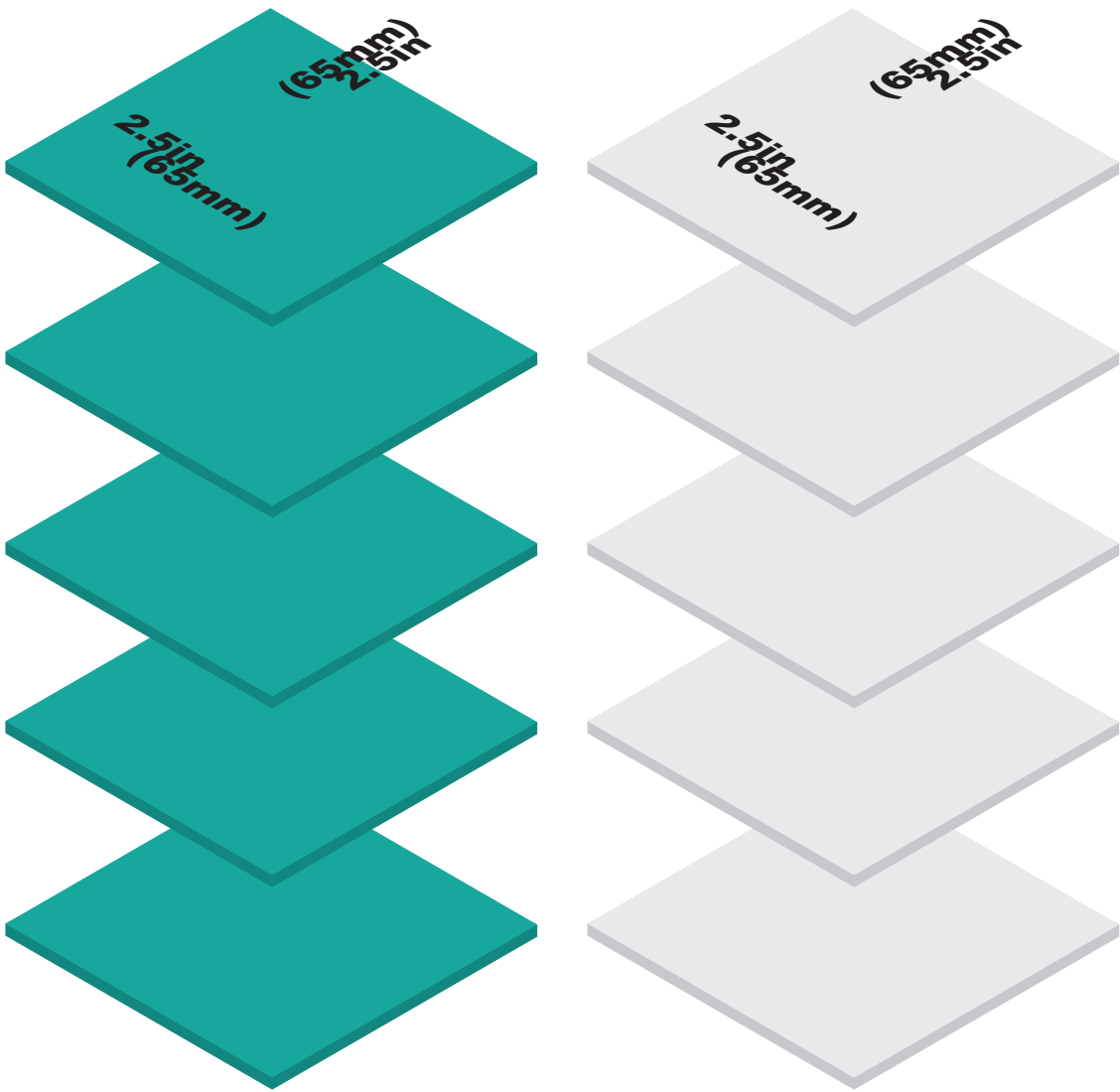


Flowmark Filters

- 1 Start with at least 5 precut mask filter squares.

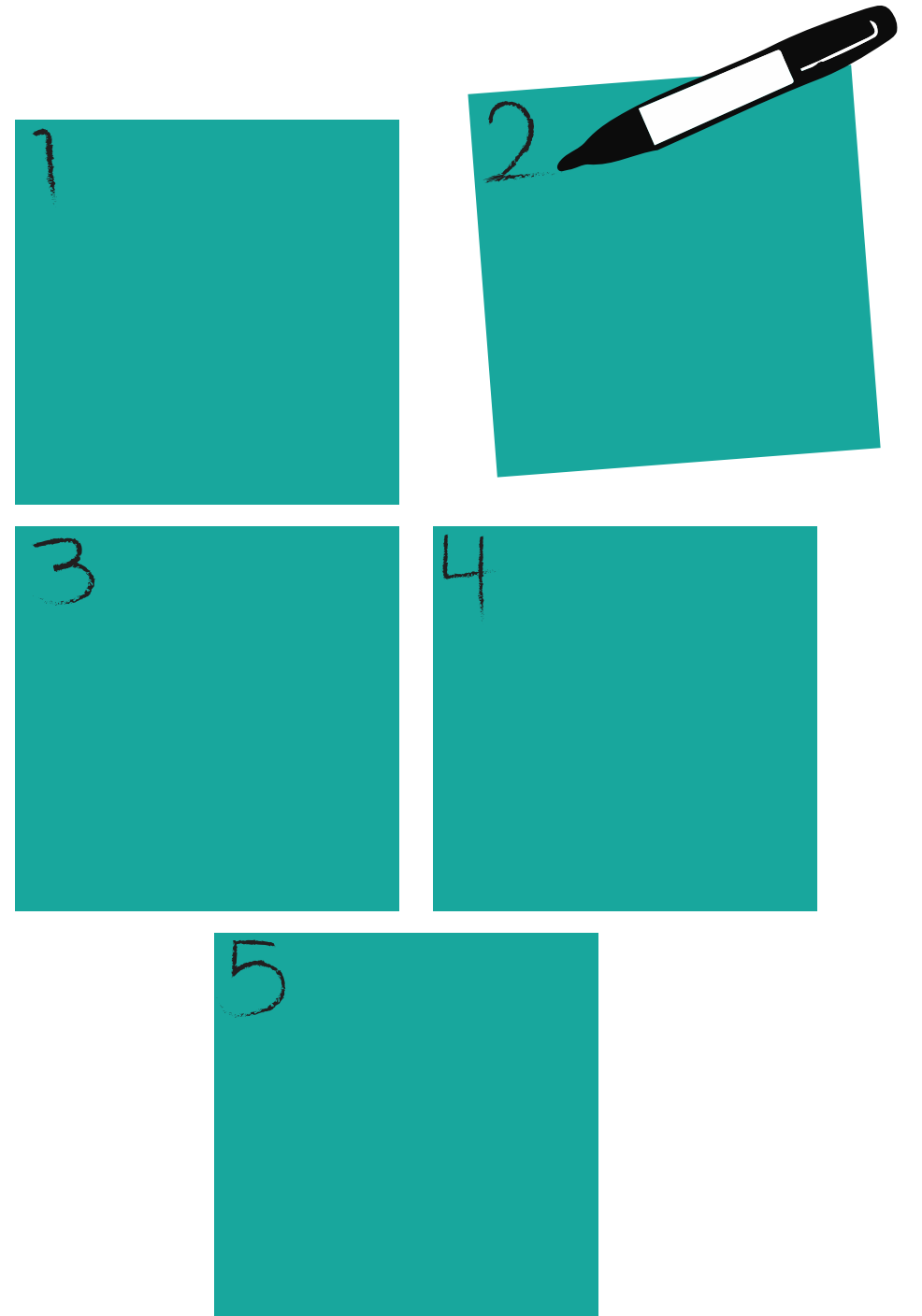


Filter Material Numbering

- 2 Number each square as 1,2,3, etc. in the corner. This will be your daily filter change schedule.

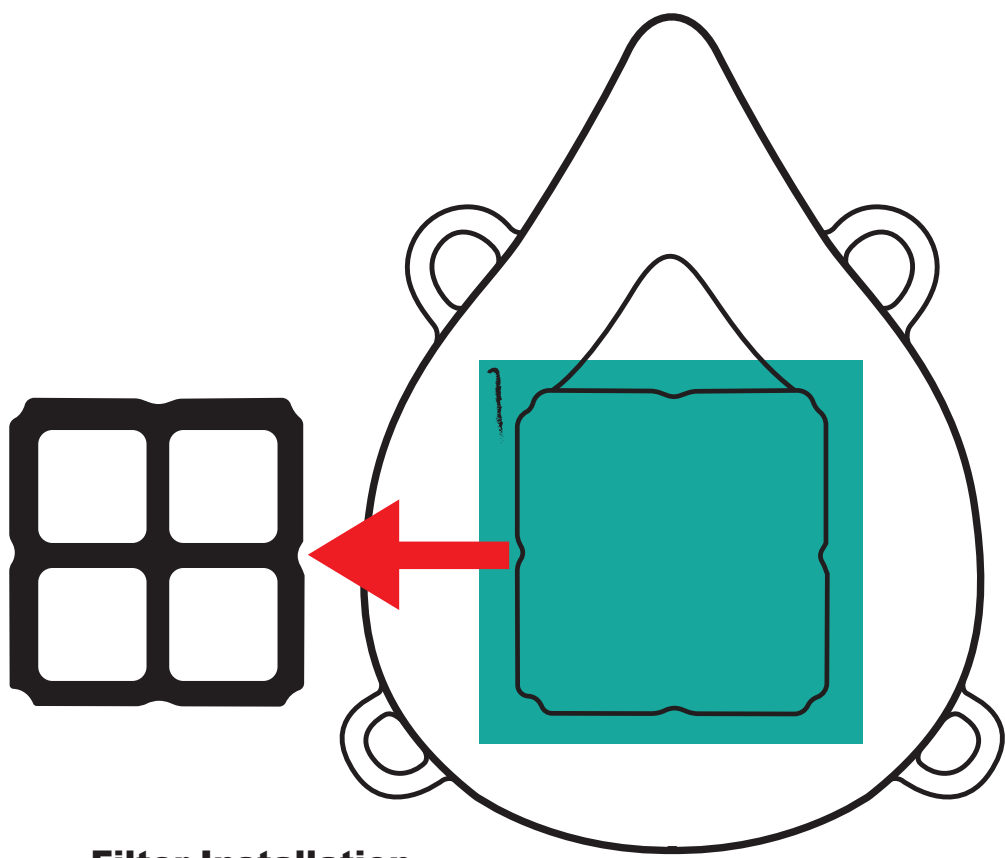


Filter Installation

- 3 Remove the filter frame and place your filter over the hole.

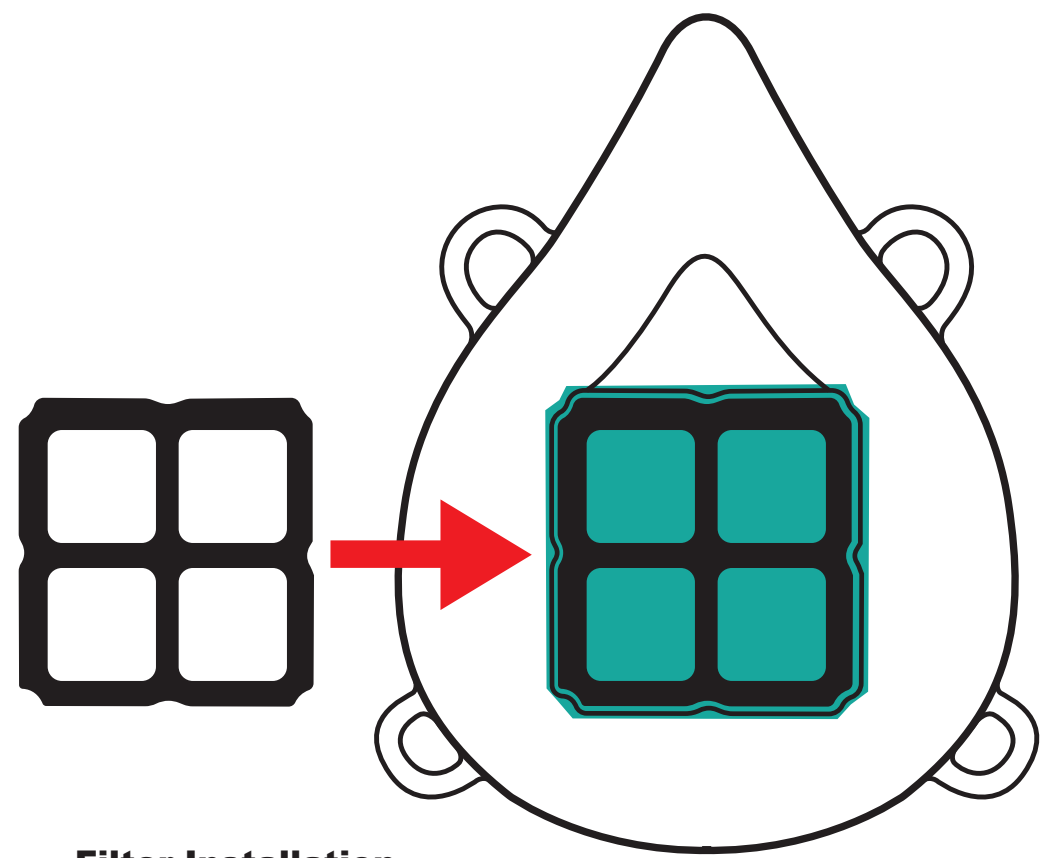



Filter Installation

- 4 Press the filter frame into the mask pressure fitting your filter into place.

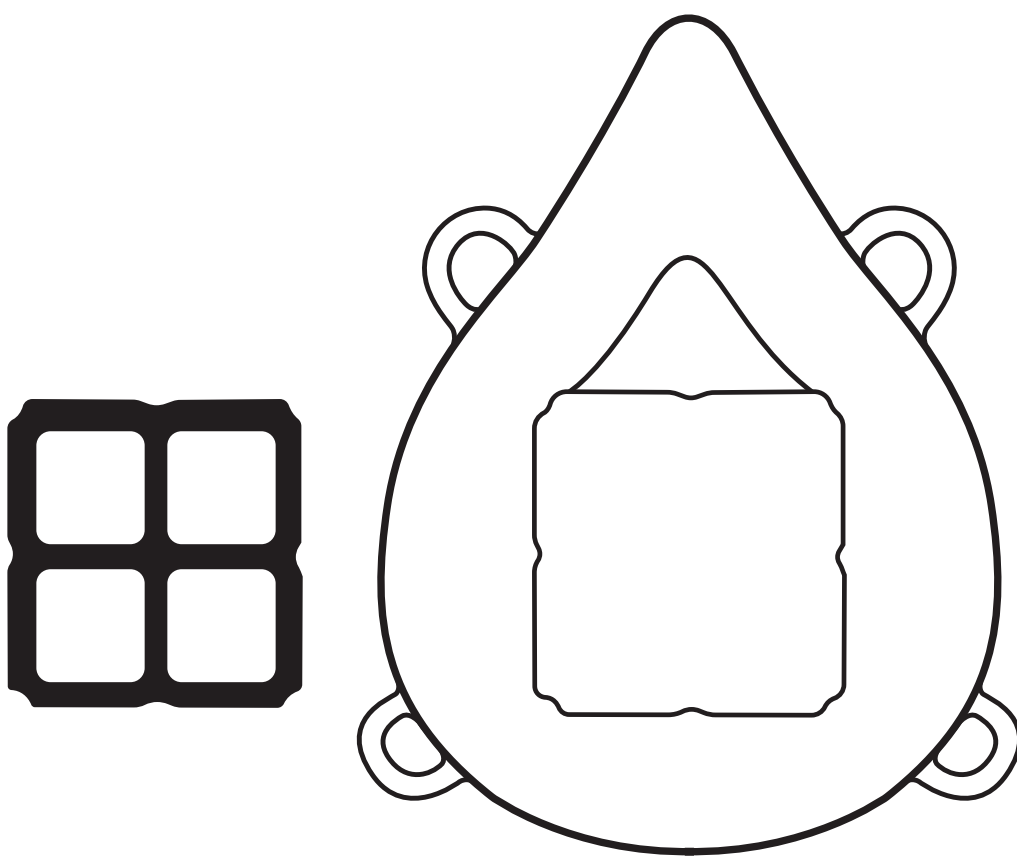

Daily Filter Removal

- 5 When the day is done, remove your filter and place it individually into closed a paper bag. Do not use a plastic bag.

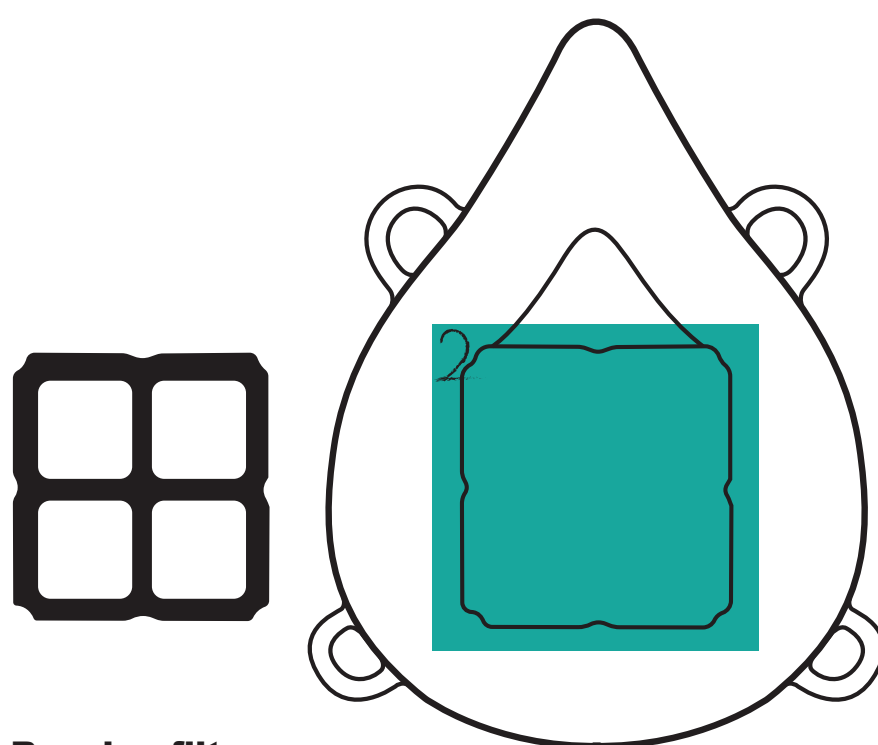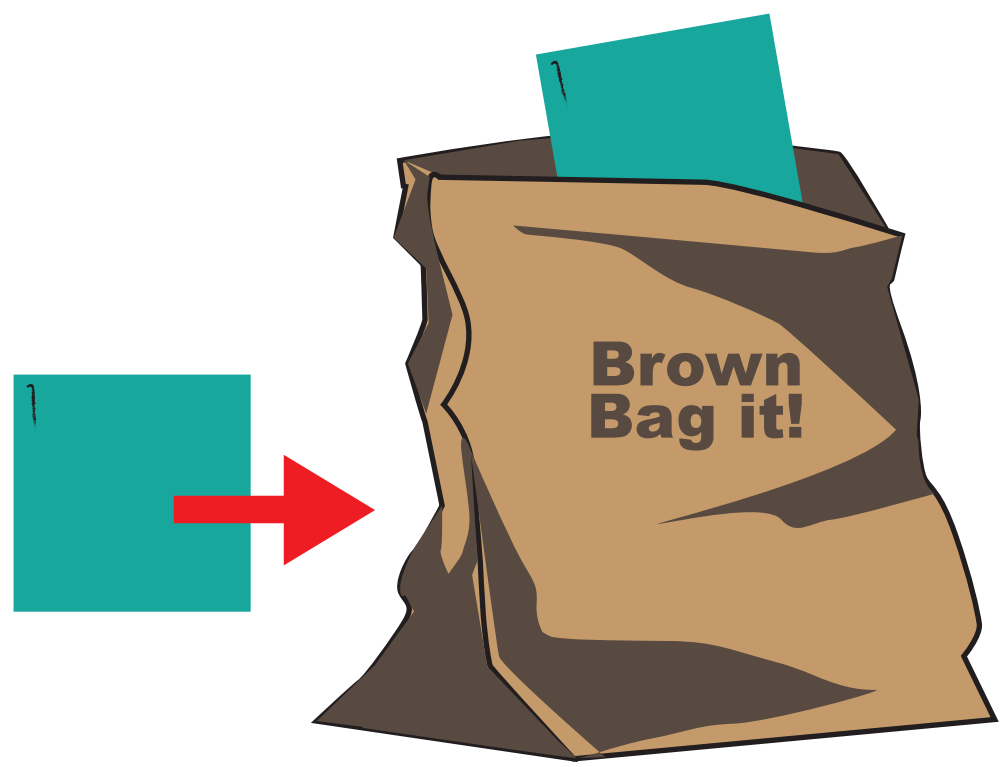

Reusing filters

- 6 Repeat steps 3-6 **every day** using a new filter each day until it becomes necessary to reuse your filters starting at filter #1.

You must have at least five filters in your rotation, and only use one per day to comply with CDC guidelines. Do not reuse a filter in less than five days