



# Instructions for use and reuse of surgical or N95 masks with the Montana Mask

Compliant with the "Decontamination & Reuse of Filtering Facepiece Respirators" issued by the CDC. See <https://www.cdc.gov/coronavirus/2019-ncov/hcp/ppe-strategy/decontamination-reuse-respirators.html> for more information.

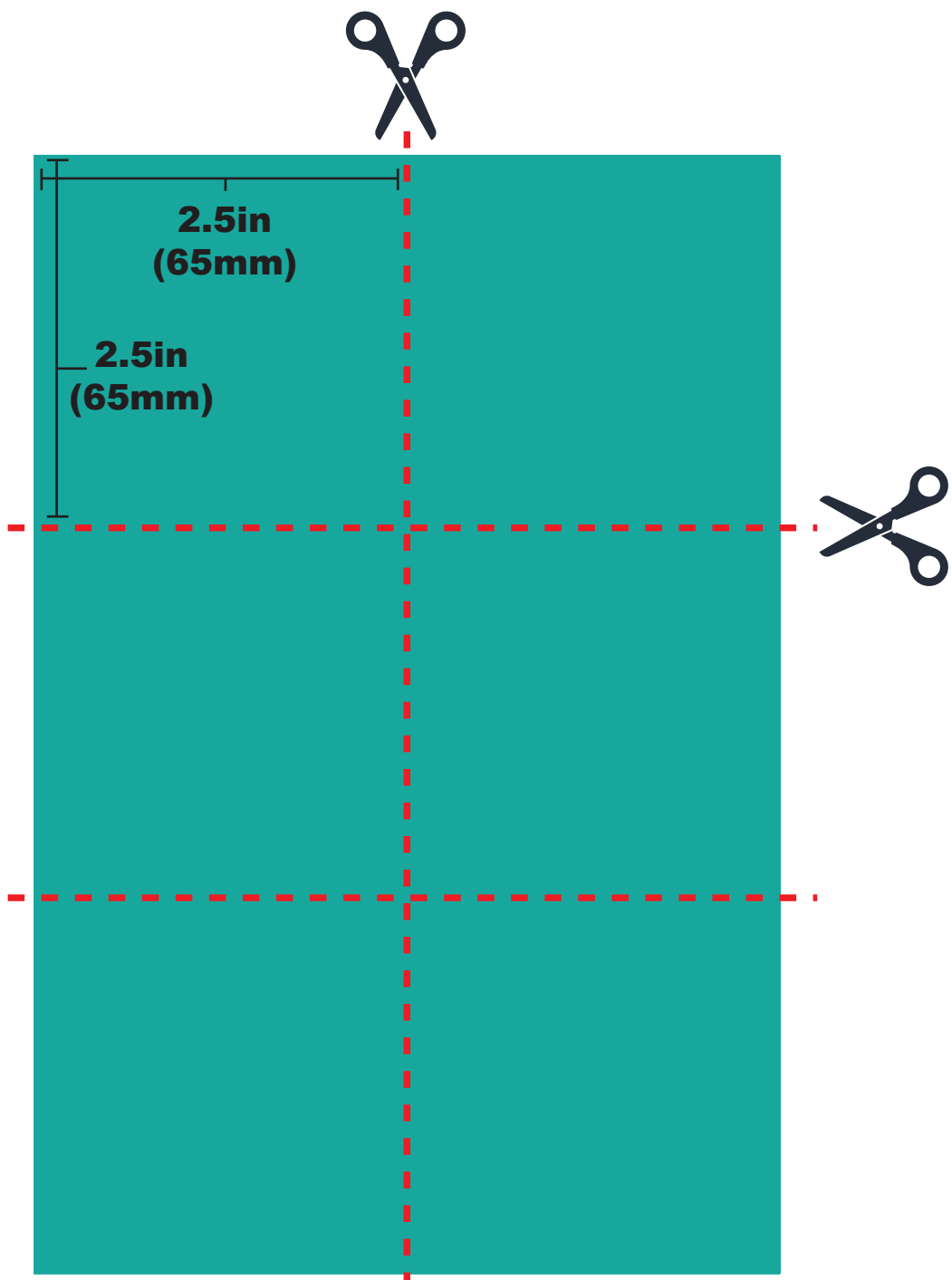


## Filter Material Cutting

- 1 Cut surgical or N95 mask in to 2.5in x 2.5in (65mm x 65mm) squares as many times as possible.

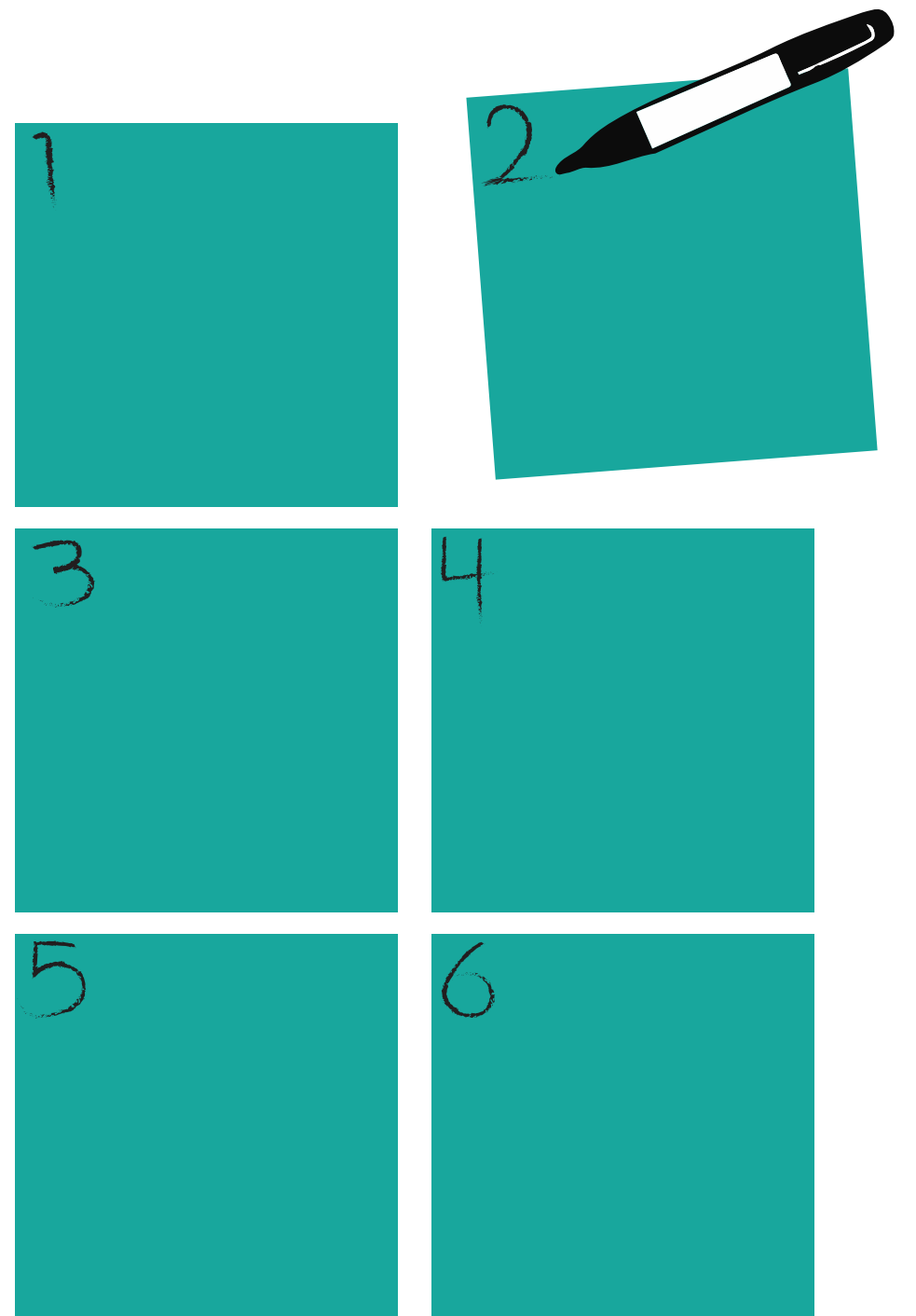


## Filter Material Numbering

- 2 Number each square as 1,2,3, etc. in the corner. This will be your daily filter change schedule. Some mask sizes may make more than 6 square filters when cut, If possible do so.

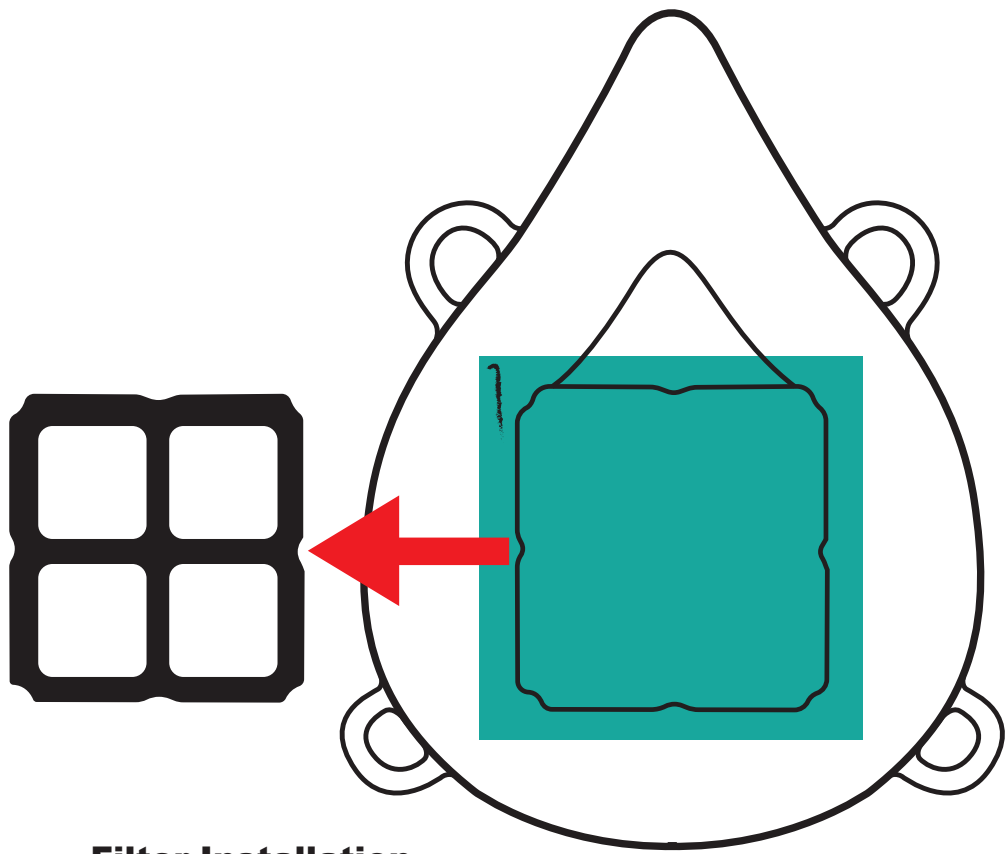


## Filter Installation

- 3 Remove the filter frame and place your filter over the hole.

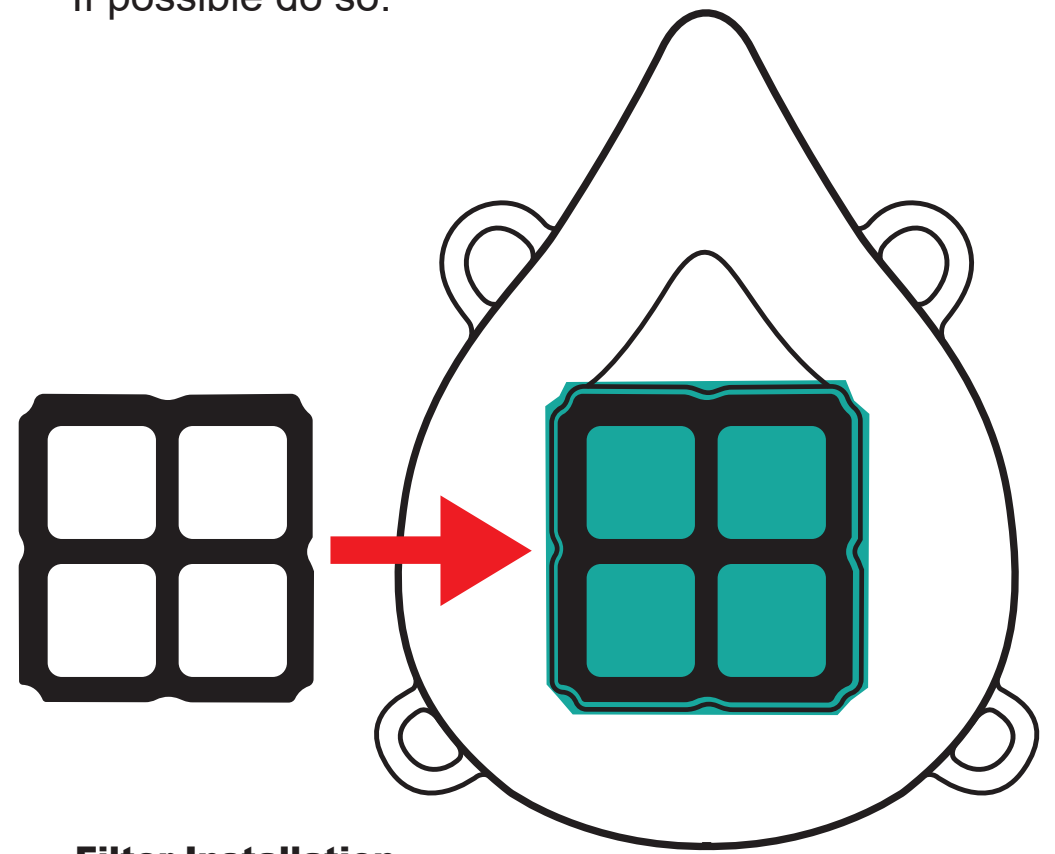



## Filter Installation

- 4 Press the filter frame into the mask pressure fitting your filter into place.

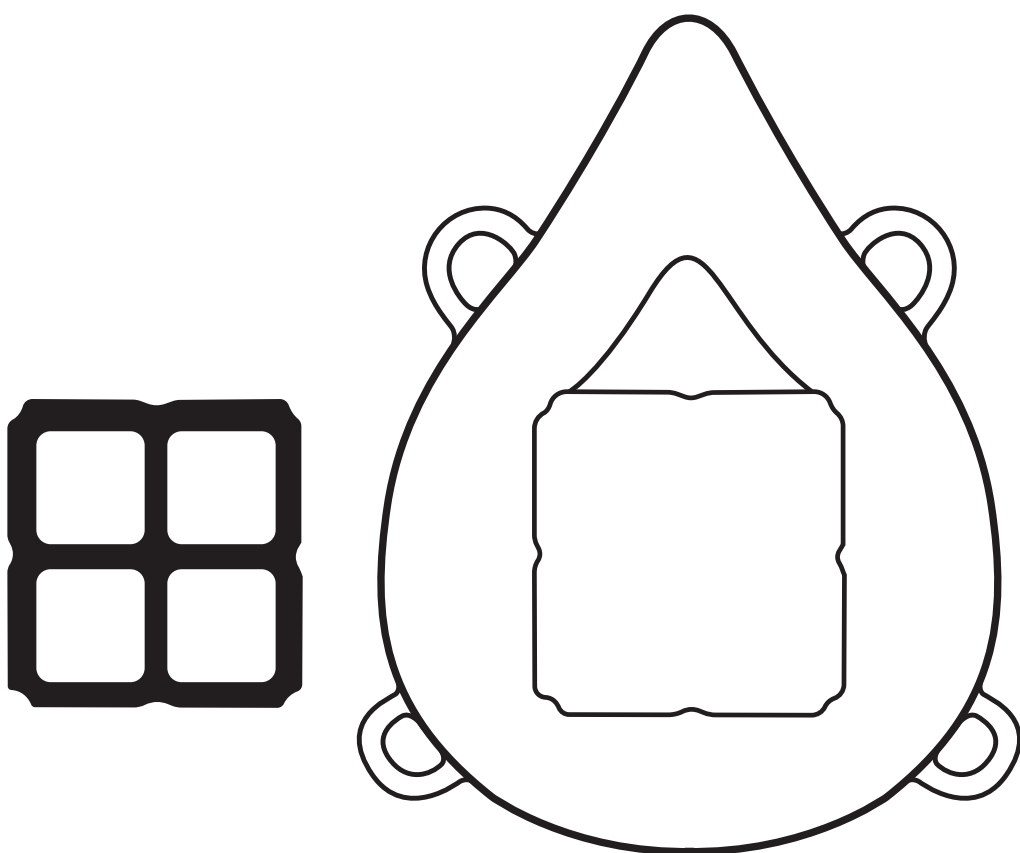

## Daily Filter Removal

- 5 When the day is done, remove your filter and place it individually into closed a paper bag. Do not use a plastic bag.

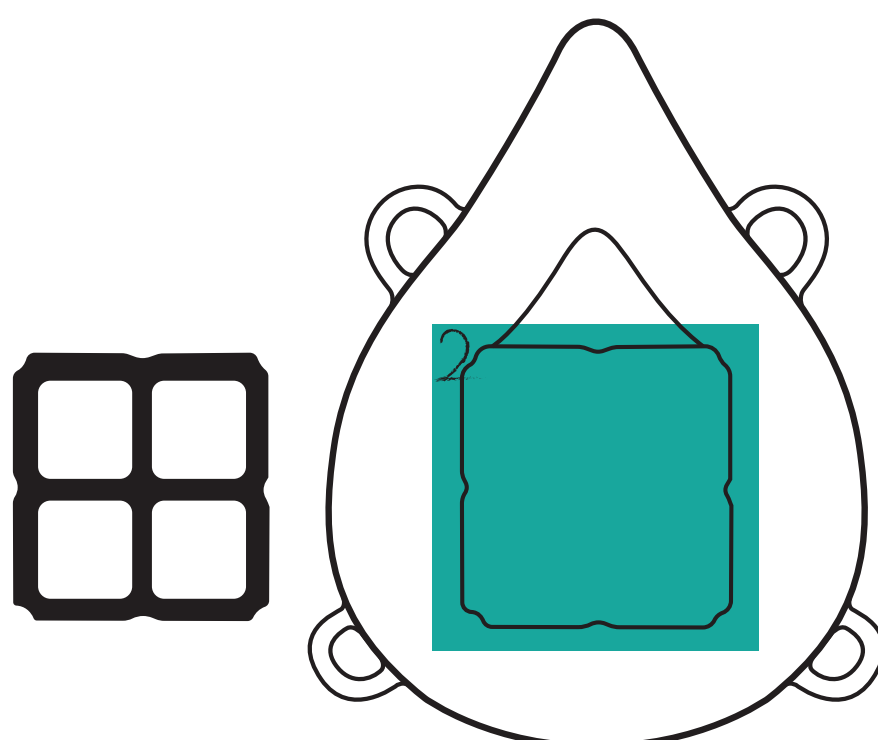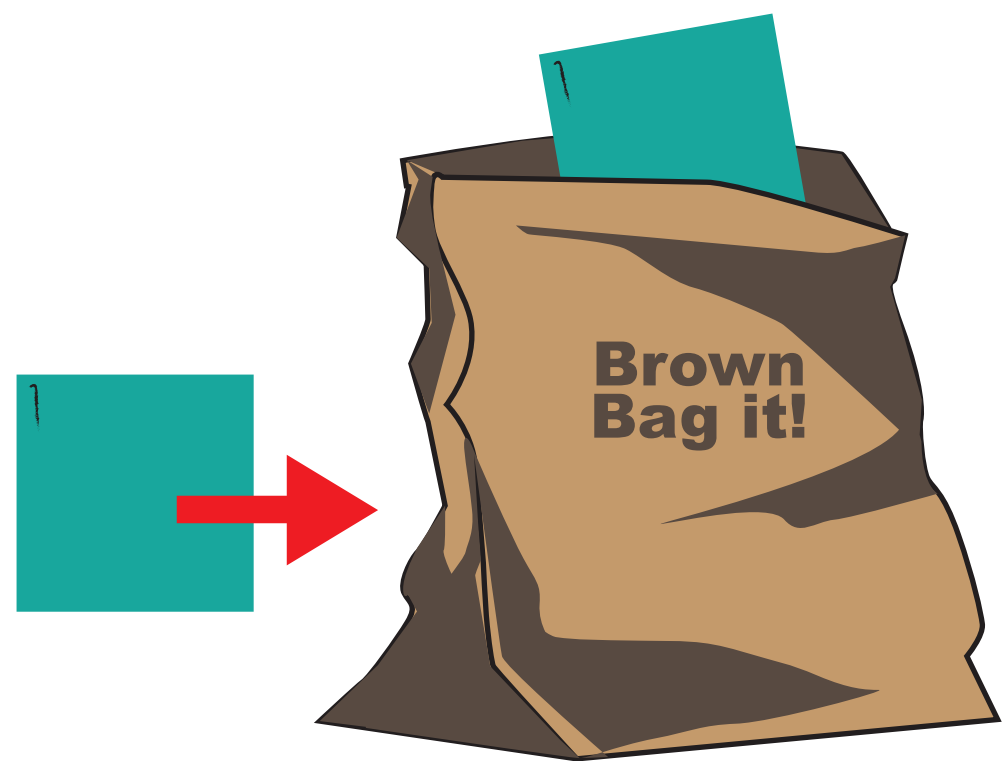

## Reusing filters

- 6 Repeat steps 3-6 **every day** using a new filter each day until it becomes necessary to reuse your filters starting at filter #1.

**You must have at least five filters in your rotation, and only use one per day to comply with CDC guidelines. Do not reuse a filter in less than five days**